

Endomag
for patients



Breast localisation with Magseed®

endomag⁺



Learn about breast localisation with Magseed®

Having a breast tumour removed	3
What is a lumpectomy or partial mastectomy?	4
How surgeons use Magseed®	4
The Magseed® procedure	5
1) Your appointment in radiology	5
2) Removing the seed during surgery	6
3) What happens after the surgery?	6
Find out more	7



Having a breast tumour removed

If you have been diagnosed with early-stage breast cancer, or have a suspicious looking lump that has been identified as a possible tumour during screening, you may have been offered breast-conserving surgery – also known as a lumpectomy or partial mastectomy.

This is a surgical treatment that aims to completely remove the tumour in one piece and leave behind as much healthy tissue as possible. This technique is less invasive

than having the whole breast removed (also known as a mastectomy), and often has better cosmetic outcomes.

Thanks to more regular screenings, breast tumours are now being identified earlier, allowing more women the option of breast conserving surgery.

This has the positive effect of improving outcomes and more women surviving breast cancer than ever before.

What is a lumpectomy or partial mastectomy?

A lumpectomy or partial mastectomy is a breast cancer treatment which involves surgically removing the tumour in one piece, as well as a thin margin of healthy tissue around it to ensure no tumour cells are left behind.

The amount of tissue that is removed will depend on the size of the tumour and the surgeon's ability to accurately locate it.



How surgeons use Magseed®

In order for the surgeon to know the exact location of the tumour, there are a number of aids that can be used to 'mark' it. One of these is the Magseed® marker.

The Magseed® marker will be placed in the centre of your tumour well in advance of your surgery. Once you are in the operating room, your surgeon will first use your mammogram images to give them a rough idea of where the tumour is located in your breast.

A probe will then be used to detect the signal from the Magseed® marker, which will enable them to accurately locate the seed. This enables the tumour to be removed accurately and completely.

1) Your appointment in radiology


With a 'Magseed® localisation' procedure, you will attend a placement appointment prior to your surgery

Your Magseed® marker could be placed many weeks before your planned surgery date, at a time that suits both you and your surgeon.

When it's time to place the marker, you'll receive a local anesthetic. The Magseed® will then be placed under ultrasound guidance in a procedure room. This is a quick and efficient procedure.

After your Magseed® has been inserted, you'll be able to carry on life as normal up to your date of surgery, without fear of discomfort or displacement.

For patients needing separate SLNB procedures associated with Mastectomies, Magtrace® will also be administered in a procedure room with local anaesthetic.

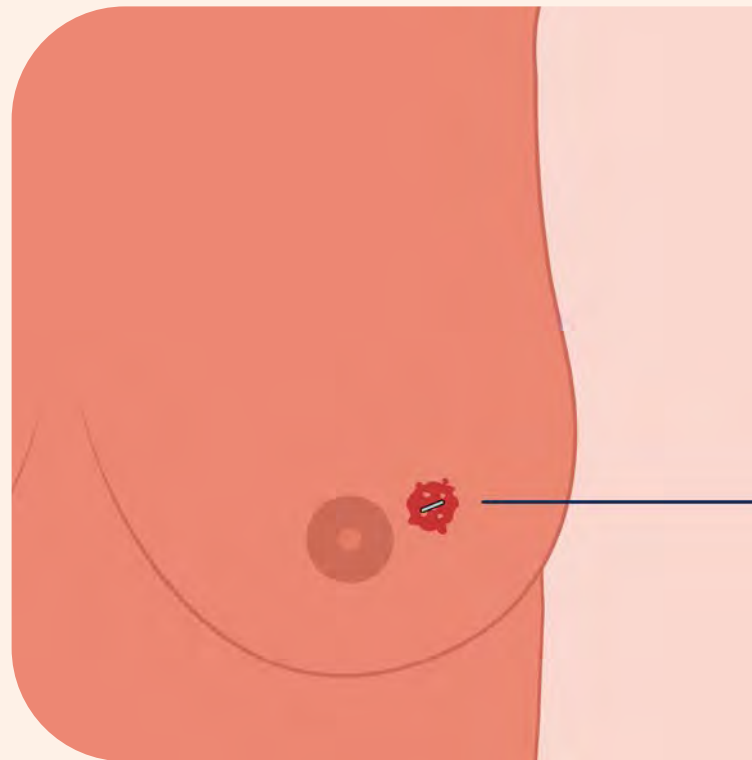

▲ *Magseed® is implanted into the tumour*

2) Removing the seed during surgery

The surgery to remove a tumour containing a Magseed® marker is a routine procedure for a breast surgeon.

With your Magseed® already placed, you will be ready to go straight through to the operating room at the agreed appointment time.

During the surgery, your surgeon will use a probe to detect and locate the previously placed Magseed® marker in your breast – with the markers magnetic signal accurately guiding them to the site of the cancer to be removed.



3) What happens after the surgery?

Once the tumour has been removed, it will be sent to pathology for closer examination, to determine whether the cancer has been completely removed.

If some of the tumour has been left behind, your surgeon will be able to remove smaller ‘shaves’ of breast tissue to ensure all of it is removed.

The likelihood of leaving any of the tumour behind with the Magseed® marker has been shown to be very low.



FIND OUT MORE

If you have any more questions about your surgery, please do ask your surgeon or the hospital team.

Thousands of surgeons and radiologists from across the world now use the Magseed® marker. You can hear our interviews with a selection of these surgeons at [endomag.com/videos](https://www.endomag.com/videos).

We also have a number of resources and stories on our website which feature more information about Magseed® at [endomag.com/patients](https://www.endomag.com/patients).

endomag⁺

Improving breast cancer
treatment for everyone.